Heartworm Antigen-Anaplasma Phagocytophylum-Plasmas-Borrelia Burgdorferi-Ehrlichia Canis-Ewingii Antibody Test Kit

For veterinary use only.

English version 

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SNAp® 4Dx® Plus Test

In vitro diagnostic for the detection of *Dirofilaria immitis* antigen, antibody to *Anaplasma phagocytophylum*, antibody to *Anaplasma platys*, antibody to *Borrelia burgdorferi*, antibody to *Ehrlichia canis*, and antibody to *Ehrlichia ewingii* in canine serum, plasma, or whole blood.

Precautions and warnings

- All wastes should be properly decontaminated prior to disposal.
- Do not mix components from kits with different serial numbers.
- Do not use a SNAP device that has been activated prior to the addition of sample.
- Mosquito heartworm infections typically produce antigen levels that are below the detection capability of this antigen test.
- Refer to country specific Material Safety Data Sheet for regional hazard identification.
- Keep out of reach of children and pets.

Storage

- Store at 2–8°C until expiration date.
- SNAP devices and reagents can be stored at room temperature (18–25°C) for 90 days or until the expiration date, whichever occurs first.
- After SNAP devices and reagents are removed from 2–8°C for more than 24 hours, the expiration date is 90 days or the printed expiration date, whichever occurs first. If the 90-day expiration date occurs prior to the printed expiration date, record the new date in the space provided on the kit.
- Keep out of reach of children and pets.

Kit components

Item	Reagents	Quantity
1	1 or 5 bottles Anti- <i>D. immitis</i> / <i>Anaplasma spp.</i> / <i>B. burgdorferi</i> / <i>E. canis</i> / <i>E. ewingii</i> : HRPO conjugate (preserved with gentamicin and ProClin™ 150)	7.0 mL
2	SNAP device	5, 15, 30, or 180
Reagents contained in each device:		
Wash solution (preserved with ProClin™ 150)		0.4 mL
Substrate solution		0.6 mL
Other components: transfer pipettes, sample tubes, and reagent rack		

WARNING
Conjugat - H317/P261/P280/P302+P332+P313: May cause an allergic skin reaction. Avoid breathing mist/vapors. Wear protective gloves. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.

Conjugat - H317/P261/P280/P302+P332+P313: May cause an allergic skin reaction. Avoid breathing mist/vapors. Wear protective gloves. IF ON EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention.

Sample information
• Samples must be at room temperature (18–25°C) before beginning the test procedure.

• Serum, plasma, or anti-coagulated whole blood (e.g., EDTA, heparin), either fresh or stored at 2–8°C for up to one week, can be used.

• For longer storage, serum, or plasma can be frozen (–20°C or colder) and then reconstituted before use.

• Hemolyzed or lipemic samples will not affect test results.

Test procedure

1. If stored in a refrigerator, allow all components to equilibrate to room temperature (18–25°C) for 30 minutes before use. **Do not heat.**

2. Using the pipette provided, dispense **3 drops of conjugate** to the sample tube.

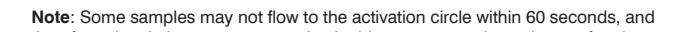
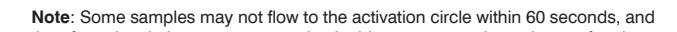
3. Holding the bottle vertical, add **4 drops of conjugate** to the sample tube.

4. Cap the sample tube and mix it thoroughly by **Inverting 3–5 times**.

5. Place the device on a horizontal surface. Add the entire contents of the sample tube to the sample well, being careful not to splash the contents outside of the sample well.

The sample will flow across the result window, reaching the activation circle in 30–60 seconds. Some sample may remain in the sample well.

6. When color EBSI appears in the activation circle, push the activator firmly until it is flush with the device body.

 Right
 Wrong

Note: Some samples may not flow to the activation circle within 60 seconds, and therefore, the circle may not turn color. In this case, press the activator after the sample has flowed across the result window.

7. Read result at **8 minutes**.

Note: The positive control may develop sooner, but results are not complete until 8 minutes.

Interpreting the test results

Positive result

Any color development in the sample spots indicates the presence of *Dirofilaria immitis* antigen, *A. phagocytophylum* antibody, *A. platys* antibody, *B. burgdorferi* antibody, *E. canis* antibody, or *E. ewingii* antibody in the sample.

Notes:

- The *A. phagocytophylum/A. platys* spot cannot differentiate between the two species; a positive result indicates presence of antibodies to *A. phagocytophylum* and/or *A. platys*.
- The *E. canis/E. ewingii* spot cannot differentiate between the two species; a positive result indicates presence of antibodies to *E. canis* and/or *E. ewingii*.
- In a low percentage of samples (0.027% as reported), interfering substances in the patient's blood can cause all spots on the device to react as positive. In this event, the sample should be re-assayed as plasma or serum to reduce the likelihood of interference.

Negative result

Only positive control spot develops color.

Invalid results

Background—if the sample is allowed to flow past the activation circle, background color may result. Some background color is normal.

However, if colored background obscures test result, repeat the test.

No color development—if the positive control does not develop color, repeat the test.

IDEXX SNAPshot Dx® analyzer

Test results can also be read using the SNAPshot Dx® analyzer. A complete description of how to enter patient data and read test results using the SNAPshot Dx can be found in the SNAPshot Dx analyzer operator's guide.

Borrelia burgdorferi vaccine cross-reactivity

—The test for *Borrelia burgdorferi* assay

detects antibodies induced as a result of natural infection with the organism and not following immunization with the following vaccines:

Recombitek® Lyme, LymeVax®, and Galaxy® Lyme and Nobivac® Lyme.

Recombitek is a registered trademark of Merial, Inc. LymeVax is a registered trademark of Pfizer, Inc. Galaxy and Nobivac are registered trademarks of Merck Animal Health.

IDEXX SNAPshot Dx® analyzer

Test results can also be read using the SNAPshot Dx® analyzer. A complete description of how to enter patient data and read test results using the SNAPshot Dx can be found in the SNAPshot Dx analyzer operator's guide.

Borrelia burgdorferi vaccine cross-reactivity

—The test for *Borrelia burgdorferi* assay

detects antibodies induced as a result of natural infection with the organism and not following immunization with the following vaccines:

Recombitek® Lyme, LymeVax®, and Galaxy® Lyme and Nobivac® Lyme.

Recombitek is a registered trademark of Merial, Inc. LymeVax is a registered trademark of Pfizer, Inc. Galaxy and Nobivac are registered trademarks of Merck Animal Health.

Symbol descriptions

 Use by date	 Authorized representative in the European Community
 LOT	Batch code (lot)
 SN	Serial number
 Temperature limitation	 Consult instructions for use
 Manufacturer	 In vitro diagnostic
 REF	Catalog number

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Information about the brevet: idexx.com/patents.

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© 2023 IDEXX Laboratories, Inc. Tous droits réservés.