SNAP* Cortisol Test Kit

For use only with the IDEXX SNAPshot Dx* Analyzer.

For veterinary use only.

English version

Overview

The SNAP Cortisol Test Kit is an enzyme-linked immunosorbent assay (ELISA) for the quantitative measurement of cortisol in dog serum. For in vitro use only.

SNAP Cortisol test ranges

Dynamic ranges:

- 2.5–30 µg/dL (69–828 nmol/L) for post-ACTH stimulation, Cushing’s-suspected dogs and therapeutic monitoring > 10 µg/dL (>276 nmol/L)
- 0.5–10 µg/dL (14–276 nmol/L) for all other tests

⚠️ Precautions and warnings

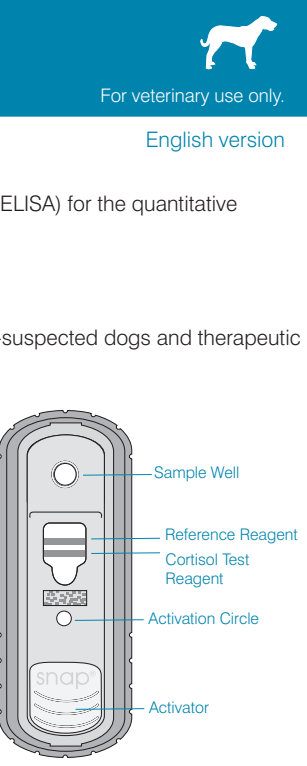
- This test cannot be visually interpreted. The IDEXX SNAPshot Dx analyzer must read all tests.
- Use a separate pipette tip and sample tube for each test.
- Do not use components after their expiration dates.
- Do not mix components from different lots of test kits.
- Do not use a device that does not have two blue stripes visible before adding the sample.
- Keep the device in its sealed foil pack at room temperature (18–27°C/64–80°F) for at least 10 minutes before use.
- Prewarming of the conjugate solution is not required.
- IMPORTANT:** Do NOT use the SNAP Cortisol test if the patient has received:
 - Dexamethasone (doses >0.1mg/kg) in the last 7 days.
 - OR**
 - Prednisone or other corticosteroids in the past 4 weeks.

Storage

- SNAP devices and test reagents must be stored at 2–8°C (36–46°F).
- Do not heat or microwave the kit components.
- Take out only the kit components you will need.
- Do not leave kit components at room temperature for more than 8 consecutive hours.

Kit components

- Conjugate solution
- SNAP test devices
- Disposable pipette tips for 300-µL manual pipettor
- Disposable pipette tips for the 25-µL and 100-µL manual pipettors
- Disposable sample tubes
- Workstation

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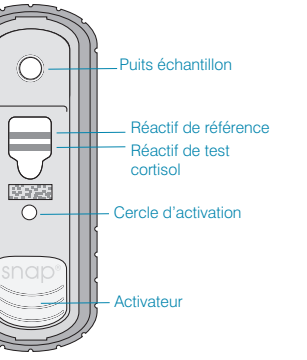
⚠️ Componentes required (not provided in kit)

The following required components are provided with the SNAPshot Dx analyzer:

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Sample information

- Do not use plasma or whole blood. Use serum, either fresh, previously frozen or stored at 2–8°C (36–46°F) for up to 7 days. To store longer than 7 days, freeze the sample (-20°C/-4°F or below). Remove serum from the red blood cells before freezing. Bring previously refrigerated or frozen samples to room temperature 18–27°C (64–80°F) before running assays.
- SNAP cortisol sample size requirements:
 - 25 µL for post-ACTH stimulation, Cushing’s-suspected dogs and therapeutic monitoring > 10 µg/dL (>276 nmol/L)
 - 100 µL for all other tests
- Centrifuge previously frozen or older samples before use.
- Results of this test are not affected by hemolysis or lipemia.

⚠️ Test procedure

The SNAPshot Dx analyzer provides on-screen instructions throughout the analysis. For complete instructions, refer to the Using the SNAPshot Dx Analyzer chapter in the SNAPshot Dx Operator’s Manual.

IMPORTANT:

- Keep the SNAP device horizontal during the entire test procedure.
- Watch carefully for **sample or blue color in the activation circle**.
- Depress the activator when color **FIRST** appears in the activation circle.
 - Note: **Delay in activation could influence results**
 - Note: Some samples may not flow to the activation circle and, therefore, the circle may not turn color. In this event, please contact IDEXX Technical Support.
- Push the activator firmly until it is flush with the device body.
- Insert the device into the analyzer immediately after activation.

Test	Sample Volume	Conjugate Volume
Baseline Cortisol	100 µL	300 µL
Post-Dexamethasone Cortisol (post low- or high-dose dexamethasone)	100 µL	300 µL
Post-ACTH Stim Cortisol: Cushing’s Suspected	25 µL	300 µL
Post-ACTH Stim Cortisol: Addison’s Suspected	100 µL	300 µL
Post-ACTH Stim Cortisol: Therapeutic Monitoring	100 µL	300 µL
Post-ACTH Stim Cortisol: Therapeutic Monitoring > 10 µg/dL	25 µL	300 µL

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Interpreting test results

The recommendations below are intended to provide general guidance only. As with any diagnosis, you should use clinical discretion with each patient based on a complete evaluation of the patient, including history, physical examination and complete laboratory profile.

NOTE: Dexamethasone (doses >0.1mg/kg), prednisone and other corticosteroids are expected to artificially increase the reported result.

A. ACTH stimulation test

Pre-ACTH	Interpretation	Post-ACTH	Interpretation
<2 µg/dL	Must evaluate in conjunction with post-ACTH result. If both results are <2 µg/dL, results are consistent with hypoadrenocorticism. Begin treatment with mineralocorticoid and/or glucocorticoid as appropriate.	<2 µg/dL	Ideally, should be evaluated in conjunction with pre-ACTH result. If both results are <2 µg/dL, results are consistent with hypoadrenocorticism. Begin treatment with mineralocorticoid and/or glucocorticoid as appropriate.
2–6 µg/dL	Normal	2–6 µg/dL	Inconclusive
>22 µg/dL	Consistent with Cushing’s syndrome; perform high-dose dexamethasone suppression.	6–18 µg/dL	Normal
		18–22 µg/dL	Equivocal; Cushing’s syndrome possible
		>22 µg/dL	Consistent with Cushing’s syndrome; perform high-dose dexamethasone suppression to discriminate between PDH and ATH. ACTH level and/or abdominal ultrasound.

B. Low-dose dexamethasone suppression test

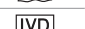
4-hour cortisol level	8-hour cortisol level	Interpretation
Not necessary	<1 µg/dL	Normal
1–1.5 µg/dL	1–1.5 µg/dL	Inconclusive, consider repeating in 8–12 weeks
>1.5 µg/dL and >50% of baseline	>1.5 µg/dL and >50% of baseline	Consistent with Cushing’s syndrome; perform high-dose dexamethasone suppression, endogenous ACTH concentration and/or abdominal ultrasound to discriminate between PDH and ATH.
<1.5 µg/dL or <50% of baseline	>1.5 µg/dL and >50% of baseline	Consistent with PDH
>1.5 µg/dL or >50% of baseline	>1.5 µg/dL and <50% of baseline	Consistent with PDH
<1.5 µg/dL or <50% of baseline	<1.5 µg/dL and <50% of baseline	Consistent with PDH

C. High-dose dexamethasone suppression test

NOTE: Take care to not exceed 0.1 mg/kg of dexamethasone.

4-hour cortisol level	8-hour cortisol level	Interpretation
<1.5 µg/dL or <50% of baseline	>1.5 µg/dL and >50% of baseline	Consistent with PDH
>1.5 µg/dL and >50% of baseline	<1.5 µg/dL and <50% of baseline	Consistent with PDH
<1.5 µg/dL or <50% of baseline	<1.5 µg/dL or <50% of baseline	Consistent with PDH
>1.5 µg/dL and >50% of baseline	>1.5 µg/dL and >50% of baseline	Additional testing required to differentiate PDH from ATH; consider measuring plasma ACTH levels and/or performing an abdominal ultrasound.

<div>⚠️</div> <div>WARNING</div>	Conjugate / Wash Solution - H317/H412/P261/P280/P333+P313: May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention.
---	---

Symbol Descriptions			
	Use by date		Authorized Representative in the European Community
	Batch Code (Lot)		Consult instructions for use
	Catalog Number		In vitro diagnostic
	Temperature limitation		Major change in the user instructions
	Manufacturer		
IDEXX Technical Support			
USA/Canada: 1 800 248 2483 • idexx.com			
Australia: 1300 44 33 99 • idexx.com.au			
Europe: idexx.eu			
<i>If possible, please have your kit lot number available.</i>			
<p><small>*Valid only in countries where mitotane is approved for use; trilostane may be used as an alternative where approved for use, in accordance with applicable protocols.</small></p> <p><small>*SNAP and SNAPshot Dx are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.</small></p> <p><small>All other products and company names are trademarks of their respective holders.</small></p> <p><small>Patent information: idexx.com/patents</small></p> <p><small>© 2022 IDEXX Laboratories, Inc. All rights reserved • 06-03312-14</small></p>			

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© 2022 IDEXX Laboratories, Inc. Alle Rechte vorbehalten.